

PRESEPT® DISINFECTANT TABLETS 2.5g (CODE MPR021)

DESCRIPTION

Each PRESEPT® Disinfectant Tablet contains 2.5g of sodium dichloroisocyanurate in an inert effervescent base. PRESEPT® Disinfectant Tablets are stable and compact in storage, ensuring full biocidal activity up until the moment of use. Preparation is safe, simple, and accurate.

INDICATION

For the preparation of a wide spectrum disinfectant solution effective against vegetative bacteria, fungi, viruses and bacterial spores for disinfection of hard surfaces.

In addition, efficacy has been demonstrated in the following organisms:

Proven efficacy against:			
<i>Clostridium difficile</i>	Methicillin Resistant <i>Staphylococcus aureus</i> (MRSA)	Canine Parvovirus	Feline Panleukopenia virus

DIRECTION FOR USE

- All surfaces should be pre-cleaned prior to disinfection using PRESEPT® Solution
- Dissolve 4 tablets of 2.5g PRESEPT® Disinfection Tablets in 1 litre of water for a concentration of 5000ppm
- Wipe down surfaces with PRESEPT® Solution saturated disposable cloth, or
- If soaking equipment, ensure equipment is pre-cleaned and rinsed prior to immersion. Following disinfection, remove equipment and rinse thoroughly with potable water. Rough dry.
- For effective disinfection, PRESEPT® Solution needs to remain on the surface for 5 minutes.
- All solutions should be prepared fresh daily.



DANGER

Contact with acids liberates toxic gas. Causes serious eye irritation. May cause respiratory irritation. Very toxic to aquatic life with long lasting effects. Avoid



breathing dust/fume/gas/mist/vapours/spray.

Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. Call a POISON CENTER or doctor if you feel unwell. If eye irritation persists: Get medical attention.

CAUTION:

Avoid prolong contact with stainless steel items. PRESEPT® Tablets may be used with anionic and nonionic but not cationic detergents. Do not immerse animal fibres such as wool or silk due to strong proteolytic action. Store in cool dry place. Keep out of reach of children. Harmful if swallowed. Replace cap between uses. Contact with acids liberates toxic gases. Do not breathe dust. Avoid skin and eye contact. If eye contact occurs, irrigate with copious amounts of water. Do not use broken or chipped tablets. Do not dissolve tablets in a closed container.


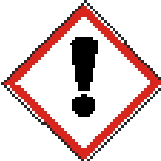
Johnson & Johnson MEDICAL PRODUCTS

Markham, Ontario L3R 0T5

© Ethicon, Inc. 2009-2015

949-581-5799

PRESEPT Tablets 2.5g Outer label (100 Tablets container)

<p>READ ENCLOSED DIRECTIONS BEFORE USE</p> <p>DESCRIPTION PRESEPT® Tablets are effervescent disinfectant tablets containing the active ingredient Sodium Dichloroisocyanurate. They dissolve rapidly in water to produce a rapid acting, wide spectrum disinfectant solution effective against vegetative bacteria, fungi, viruses and bacterial spores.</p> <p>USES Solutions prepared from PRESEPT® Tablets are suitable for the disinfection of hard surfaces. In addition, efficacy has been demonstrated in the following organisms:</p> <ul style="list-style-type: none"> • <i>Clostridium difficile</i> • Methicillin Resistant <i>Staphylococcus aureus</i> (MRSA) • Canine Parvovirus • Feline Panleukopenia virus <p>PREPARATION To prepare a disinfectant solution of 5000ppm, add four 2.5g PRESEPT® Tablet to 1L of water. For more specific dilution instructions refer to enclosed sheet.</p> <p>DANGER Contact with acids liberates toxic gas. Causes serious eye irritation. May cause respiratory irritation. Very toxic to aquatic life with long lasting effects. Avoid breathing dust/fume/gas/mist/vapors/spray. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present</p>	<p>2.5 g</p>   <p>Danger</p>	<p>PRESEPT® MPR021 DISINFECTANT TABLETS 2.5g/100 100 TABLETS each containing 2.5g SODIUM DICHLOROISOCYANURATE</p> <p>French equivalent</p> <p>DIN 01946390</p> <p>Johnson & Johnson Medical Products Markham, Ontario L3R 0T5 © Ethicon, Inc. 2001- 2015 949-581-5799</p>	<p>French equivalent of left panel</p>
---	--	--	--

and easy to do. Continue rinsing. Call a POISON CENTER or doctor if you feel unwell. If eye irritation persists: Get medical attention.

CAUTION

Harmful if swallowed. Avoid skin and eye contact. If eye contact occurs, irrigate with copious amounts of water. Do not use broken or chipped tablets. Store in a cool dry place. Do not mix with acids or cationic detergents. Do not breathe dust. Do not dissolve tablets in a closed container.